

## Dilution instructions for PRESEPT\* Disinfectant Tablets

Disinfection of	Required concentration of available chlorine	Dilution Rates			Additional Instructions
		0.5g tablets	2.5g tablets	5.0g tablets	
Blood spillage †	10,000 ppm	18 tablets in 0.5L water	7 tablets in 1L water	9 tablets in 2.5L water	Pour over blood. Using gloves, wipe up with disinfectant-saturated disposable cloth.
Pipette jars †	2,500 ppm	9 tablets in 1L water	9 tablets in 5L water	9 tablets in 10L water	Drop tablets into water-filled pipette jar. Discard daily ‡
General laboratory/ environment use †	1,000 ppm	4 tablets in 1L water	4 tablets in 5L water	3.5 tablets in 10L water	Wipe down surfaces with disposable disinfectant-saturated cloth.
Baby bottles/teats, utensils, porcelain, glassware, rubber and plastic tubing	140 ppm	1 tablets in 2L water	1 tablets in 10L water	1 tablets in 20L water	Immerse for 1 hr.
Eating utensils and crockery	140 ppm	1 tablets in 2L water	1 tablets in 10L water	1 tablets in 20L water	Rinse for 1-2 mins.
Soiled linen	140 ppm	1 tablets in 2L water	1 tablets in 10L water	1 tablets in 20L water	Immerse for 1 hr. prior to washing.
Infected linen	140 ppm	1 tablets in 2L water	1 tablets in 10L water	1 tablets in 20L water	As above
Work surfaces cubboards, floors, etc.	140 ppm	1 tablets in 2L water	1 tablets in 10L water	1 tablets in 20L water	Wash down
Dishtcloths, mops, etc.	60 ppm	1 tablets in 4.6L (1gal) water	1 tablets in 23L (5gal) water	1 tablets in 46L (10gal) water	Soak to bleach clean and deodorise

† Howie Code requirements for laboratory use. ‡ A 1% compatible detergent should also be added.

## PRESEPT\* 消毒片\* 使用說明書

消毒範圍 (Blood Spillage)	需要濃度	開水比例			使用方法
		0.5g/每片	2.5g/每片	5.0g/每片	
消毒瓶 (Pipette jars)	2500 ppm	9片/ 1公升水	9片/ 5公升水	9片/ 10公升水	將數量瓶放入必殺菌液中，有效期為24小時 以抹布浸濕擦拭表面
實驗室、環境衛生等	1000 ppm	4片/ 1公升水	4片/ 5公升水	3.5片/ 10公升水	浸泡1小時
奶瓶/壺、瓷器製品、玻璃製品、橡皮、塑膠製品	140 ppm	2公升水	10公升水	20公升水	浸泡1-2分鐘
飲食器具、調器浴、調衛生設備	140 ppm	2公升水	10公升水	20公升水	清洗前先行浸泡1小時
航機床單、衣物	140 ppm	2公升水	10公升水	20公升水	同上
感染床單或衣物	140 ppm	2公升水	10公升水	20公升水	擦拭即可
工作檯、櫃檯、地板等	140 ppm	2公升水	10公升水	20公升水	浸漬去污、除臭、殺菌一次完成
抹布、地毯等	60 ppm	4.6公升水	23公升水	46公升水	

# PRESEPT\* 消毒片\*

**PRESEPT\***  
DISINFECTANT TABLETS  
make a better  
disinfection  
**SURGIKOS**  
LIMITED  
a Johnson & Johnson company

**Record Codes:**  
SPB05, 0.5g active ingredient, 6x600 tabs.  
SPB052, 0.5g active ingredient, 12x250 tabs  
SPB053, 0.5g active ingredient, 12x72 tabs  
SPR25, 2.5g active ingredient, 6x100 tabs  
SPR25, 2.5g active ingredient, 10x40 tabs  
SPW50, 5.0g active ingredient, 6x50 tabs  
Surgikos Limited, Kirkton Campus,  
Livingston, West Lothian, EH54 7AT  
Tel: (0506) 413441 Telex 727622  
© Surgikos Limited 1985 \*Trademark

 **healthy  
medical**  
健本醫療用品有限公司